




The **TICK** 
PA+ROL
Tick Remover

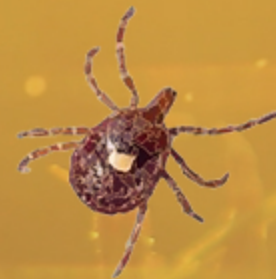

How to use:
***3 Simple Steps
to Stop Ticks
in Their Tracks:***

1. Place The Tick Patrol's tear-drop opening over the tick.
2. Slide The Tick Patrol, flat against skin, moving the tick into the smallest tapered end.
3. Pull slightly upward and away from skin. Tick is removed easily, head and all.



Why to use:

- The Tick Patrol tick remover uses natural forward leverage to remove tick from it's host, safely and effectively.
- No squirming from your children or pets because removal is done so quickly.
- No more touching, squeezing or traumatizing the tick during removal, which is when disease transfer to host occurs.

Conveniently store your Tick Patrol on a keychain, pet's collar, backpack or first aid kit.